

This fridge is for Public Health use <u>only</u>, specimen stored should be limited to:

- Rabies specimens
- Birds for West Nile virus testing
- Specimens for other animal disease situations investigated by Public Health

Specimen checklist

Before a rabies specimen is placed in this fridge, please check the following:

- ☐ Is the specimen ready for pick-up?
 - Decapitated (except bats)
 - Wrapped in absorbent paper
 - No blood visible through bag
 - o Double bagged
- ☐ Is the fridge at appropriate temperature?
- ☐ Is a full copy of the bite/bat report attached?
- ☐ Has VPH been notified that specimen is ready to go?

Thank you!

For any questions about rabies specimen preparation, please contact us at:

Los Angeles County Department of Public Health, Veterinary Public Health Program Phone: 213-989-7060; Fax: 213-481-2375; email: vet@ph.lacounty.gov

